

# Care System for Removable Dental Appliances

Recommended for all Removable Dental Appliances, including full metal denture parts, partials, orthodontic retainers, metal clasps and soft liners

## Kit includes:

Antibacterial Gel (4oz. bottle), Cleaning Solution (8 oz. bottle) and Soaking Container.



## Antibacterial Gel

Applying antibacterial gel with Zylitol to your removable dental appliances will help freshen breath and reduce the bacteria that can cause tooth decay. For best results, apply after each meal.

## Cleaning Solution

The antibacterial cleaner helps remove calcium deposits and prevents the return of stains and build-up of plaque. Safe and effective, this solution is non-corrosive to full metal denture parts, partials, orthodontic retainers, metal clasps and soft liners.



## Soaking Container

A convenient two-part container with removable drain basket allows for mixing solution, soaking your removable dental appliance and storing mixture for future use.

Review more details at [denmat.com](http://denmat.com)



**CALL: 855-8-SMILE-1**  
**brident.com**

FLY-CSK-ENG-00

**brident.**  
DENTAL & ORTHODONTICS

# Directions for Use of Care System



## Antibacterial Gel Application Instructions

**For dentures:** Squeeze a thin layer of gel directly onto the soft tissue of the denture appliance by rubbing gently into the tissue before seating the denture.

**For other removable dental appliances such as retainers or mouth guards:** Apply a small bead of gel onto the individual tooth compartments of your removable dental appliance or to the underside of the retainer before inserting into mouth. For best results apply after each meal.



*Apply a small bead of gel onto your appliance.*


## Cleaning Solution Instructions



Shake gently for 15 seconds before use.



Rinse and brush the appliance to remove any loose particles.

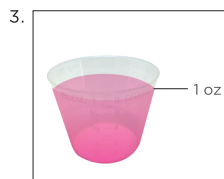

Using enclosed measuring cup, combine 1 oz. Cleaning Solution with 4 oz. water and mix together in Soaking Container.



Soak for a minimum of 4 minutes.



Rinse appliance thoroughly with clean water before placing into mouth.

**Note:** For initial use, soak overnight.

Reuse solution until cleaning ability diminishes. A mixed solution may be effective and can be utilized for up to 12 cleanings. This 8 oz. bottle can be used for up to 96 cleanings. A 16 oz. refill size is sold separately.

Review more details at [denmat.com](http://denmat.com).

Instructions have been provided for patient convenience. Please refer to the manufacturer's product guides included in their package for details. Soumava Sen DDS, P.C., Brident Dental Associates, P.C., and Brident DDS, P.C. dental practices are independently owned and operated by Soumava Sen, DDS, Yein Kim, DMD, and Sylvia Lin, DDS, licensed dentists doing business as Brident. The dental practices receive administrative and business support services from Brident Dental Services, LLC ("BDS"). BDS does not own or operate the dental practices, employ, or in any way supervise, the dentists providing dental care.