

After Braces: Retainer Care & Whitening Kit

For a Fresher & Brighter Smile

DEBAND

Retainer Care System

Kit includes:

- **Antibacterial Gel (4 oz. bottle)**
Applying antibacterial gel with Xylitol to your Removable Dental Appliance will help freshen breath and reduce the bacteria that can cause tooth decay. For best results, apply after each meal
- **Cleaning Solution (8 oz. bottle)**
The antibacterial cleaner helps remove calcium deposits and prevents the return of stains and build-up of plaque. Safe and effective, this solution is non-corrosive to orthodontic retainers
- **Soaking Container**
A convenient two-part container with removable drain basket allows for mixing solution, soaking your Removable Dental Appliance and storing mixture for future use



[Review more details at denmat.com]

OPALESCENCE GO® 10-PACK Professional Whitening

- Working from the inside out, the active ingredients penetrate your teeth to break down the discolored molecules deep down in your teeth, removing stains and whitening your smile
- Contains potassium nitrate and fluoride (shown to strengthen enamel, decrease sensitivity, and help prevent cavities)

Box Includes: 10 application tray sets (upper & lower teeth) for 10 day whitening treatment

[More details at opalescence.com]



CALL: 855-8-SMILE-1
brident.com

FLY-ORTH-DBK-ENG-BD

brident.
DENTAL & ORTHODONTICS

Directions for Use of Care System



Antibacterial Gel Application Instructions

For dentures: Squeeze a thin layer of gel directly onto the soft tissue of the denture appliance by rubbing gently into the tissue before seating the denture.

For other removable dental appliances such as retainers or mouth guards: Apply a small bead of gel onto the individual tooth compartments of your removable dental appliance or to the underside of the retainer before inserting into mouth. For best results apply after each meal.



Apply a small bead of gel onto your appliance.

Cleaning Solution Instructions



Shake gently for 15 seconds before use.



Rinse and brush the appliance to remove any loose particles.



Using enclosed measuring cup, combine 1 oz. Cleaning Solution with 4 oz. water and mix together in Soaking Container.



Soak for a minimum of 4 minutes.



Rinse appliance thoroughly with clean water before placing into mouth.

Note: For initial use, soak overnight.

Reuse solution until cleaning ability diminishes. A mixed solution may be effective and can be utilized for up to 12 cleanings. This 8 oz. bottle can be used for up to 96 cleanings. A 16 oz. refill size is sold separately

[Review more details at denmat.com]

Instructions have been provided for patient convenience. Please refer to the manufacturer's product guides included in their package for details. Soumava Sen DDS, P.C., Brident Dental Associates, P.C., and Brident DDS, P.C. dental practices are independently owned and operated by Soumava Sen, DDS, Yein Kim, DMD, and Sylvia Lin, DDS, licensed dentists doing business as Brident. The dental practices receive administrative and business support services from Brident Dental Services, LLC ("BDS"). BDS does not own or operate the dental practices, employ, or in any way supervise, the dentists providing dental care.

DENTAL & ORTHODONTICS